

24 Hr Urine Preservatives

- Jug #1:** No preservatives
Most Chemistry tests
- Jug #2:** 10 Grams Boric Acid
Citrate (Jug #2 Only)
Cortisol, Free (can also be done on Jug #7)
- Jug #6:** 5 Grams NaCO₃
Porphyrins (Uro-, Copro-, Porphobilinogen) (*Protect from Light* Jug #6 ONLY)
- Jug #7** 24 ML 50% Acetic Acid (includes instructions sheet)
Aldosterone (Performed at Mayo, can also be done on Jug #2)
Aminolevulinic Acid (ALA)
Aldosterone (Also can be done on Jug #2, Order as Misc.)
Catecholamines, Fractionated (Also can be done on Jug #2 =#12)
Metanephrines, Fractionated (Also can be done on Jug #2, #6 =#12)
Vanillylmandelic Acid (VMA) (Also can be done on Jug #2 =#12)
5-Hydroxyindoleacetic Acid (5-HIAA) (Also can be done on Jug #2, #6, + #12)
Homovanillic Acid (Also can be done on Jug #12)
- Jug #10:** No Preservative (Metal Free Mayo Brown Jug)
Arsenic (Arsenic w/Reflex can also be done on Jug #7 and #12, order as Misc.)
Arsenic Fractionated (also can be done on Jug #7 + #12)
Lead (also can be done on Jug #7 and #12)
Mercury (Also can be done on Jug #7 and #12)
Heavy Metals (Arsenic, Lead, and Mercury) (Also can be done on Jug #7 and #12)
Aluminum (Also can be done on Jug #7 + #12)
Copper (Also can be done on Jug #7 + #12)
- Jug # 12:** 10 ML 6N HCL
Histamine (Also can be done on Jug #10)